ORBIS Metal Primer

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EN **Instructions for use**

DA **Brugsanvisning**

NO **Bruksanvisning**

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MD  0482 **REF** 301874

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EN **Instructions for use**

MDP containing primer for bonding of methacrylate based dental materials

Product description

ORBIS Metal Primer is a primer to improve the adhesion between methacrylate based dental materials to titanium, non precious and precious metal alloys.

Based on innovative MDP technology ORBIS Metal Primer allows an excellent adhesion.

Indications/Intended use

Bonding of bonding of methacrylate based dental materials to metal alloys (all types)

Performance features

The performance features of the product meet the requirements of the intended use.

Contraindications

In case of known allergy to components of ORBIS Metal Primer.

Patient target group

Persons who are treated during a dental procedure.

Intended users

This medical device should only be used by professional trained dental/dental technician users.

Application for indirect restorations

Preparing the surface to be faced

Sandblast the surface to be faced with aluminium oxide (50-110 µm) at 2-3 bar pressure. Then dry with clean compressed air.

Application

Apply a uniform layer of ORBIS Metal Primer with a brush on the surface to be faced and leave on for 10 seconds. Air thin gently for 10 seconds to remove the volatile components.

Apply a methacrylate based dental material to the prepared surface according to the corresponding instructions.

Storage

Do not store above 25 °C (77 °F)! Do not use after expiry date.

ORBIS Metal Primer is highly volatile. Close bottles immediately after use.

Additional Notes/Warnings

ORBIS Metal Primer contains Acetone.

Acetone is highly volatile. Use only in good ventilated rooms.

Avoid contact with eyes and skin.

In case of contact with eyes, rinse immediately thoroughly with plenty of water and consult an ophthalmologist.

In case of skin contact wash with water.

Keep away from children!

Disposal

Disposal of the product according to local authority regulations.

Reporting obligation

Serious incidents according to the EU Medical Devices Regulation that have occurred in connection with this medical device must be reported to the manufacturer and the competent authority.

Warranty

First Scientific Dental GmbH warrants this product will be free from defects in material and manufacture. First Scientific Dental GmbH makes no other warranties including any implied warranty of merchantability or fitness for a particular purpose. User is responsible for determining the suitability of the product for user’s application. If this product is defective within the warranty period, your exclusively remedy and First Scientific Dental GmbH’s sole obligation shall be repair or replacement of the First Scientific Dental GmbH product.

Limitation of Liability

Except where prohibited by law, First Scientific Dental GmbH will not be liable for any loss or damage arising from this product, whether direct, indirect, special, incidental or consequential, regardless of the theory asserted, including warranty, contract, negligence or strict liability.

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